# beurer BY 20

**(D) Babywaage**  
Gebrauchsanleitung

**(GB) Baby scale**  
Instruction for Use

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## Dear customer,

thank you for choosing a product of our range. Our name stands for high and thoroughly checked quality-products in the field of warmth, gentle therapy, diagnosis/blood pressure, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With regards

Your Beurer-Team

## ! Important Notes

### 1. Caution

- Protect your personal scales from impact with hard objects, moisture, dust, chemicals, toiletries, liquid cosmetics, great temperature fluctuation and closeness to sources of heat (open fires, radiators).
- Repairs may only be performed by Beurer Customer Service or by accredited retailers.
- All our personal scales comply with EU guideline 89/336 + relevant supplements. Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Beurer Customer Service.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Storage: Do not place any objects on the scale when not in use.
- Strong electromagnetic fields (e.g. cell phones) may impair the accuracy of the scale.
- The scales can accept a maximum load of 44 lb (20 kg/704 oz). The weight measurement is displayed in 0.5 oz (10 g/0.02 lb) divisions.
- The scale is not intended to be used for commercial purposes.
- Remove transport safety catch.
- Place the scale on a level and solid surface; this is important for ensuring that the scale weighs correctly.

### 2. Operation

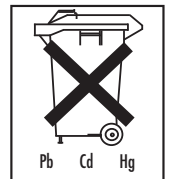
#### 2.1 Battery

The scale requires 4 1.5 V alkaline mikro AAA batteries. Before using the baby scales, these must be inserted in the battery compartment beneath the scales, ensuring the right polarity. When “” appears in the display, the batteries need to be replaced.

Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers.

Note: Batteries containing pollutant substances are marked as follows:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

## 2.2 Weighing

Weight is displayed in 10 g graduations from a minimum weight of 200 g.

Press the button  in order to switch on the scales.

The most recently stored measured value appears first. Once the display reads “0.00 kg” and “ready”, the scales are ready for the next measurement.

Now place the baby on the weighing surface. Be extremely careful not to let the baby fall and take care of the baby’s head.

These baby scales have a HOLD function. This function fixes the weight of the baby as soon as it is still enough to record an accurate measured value. If the baby moves a lot, it is impossible to determine its weight. You should therefore try to pacify the baby if it is kicking in order to do the measurement promptly. Do not hold the baby, however, as otherwise this will affect the weight reading.

As soon as the baby’s weight is fixed, the message “HOLD” appears and the weight is displayed for a further 5 seconds. The scales then automatically switch off. If you wish to save the weight, you must press “M” within this 5-second period. “M” now starts flashing instead of “HOLD”. This value is now saved and appears when you switch on the scales the next time.

If you do not want to place the baby directly on the weighing surface, but prefer to use a towel, for example, you have to place the towel on the scales BEFORE switching them on. If the towel (and any other items which you want to deduct again during weighing) weighs over 200 g, you can also use the TARA function. Press the TARA button once the items to be deducted have been weighed and “HOLD” flashes. Place the baby on the scales and continue as described above.

On delivery the scale is set at “kg” units. The button “kg/lb/oz” allows you to switch at any time between kilogramme units and pounds (lb) and ounces (oz).

## 2.3 Disposal

Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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