# HEINE mini 3000® Dermatoskop

CE 93/42/EWG/CEE/EEC

## HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202  
E-mail: info@heine.com · www.heine.com  
med 3506 2/11.08

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DEUTSCH

### HEINE mini 3000® Dermatoskop

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ENGLISH

### HEINE mini 3000® Dermatoscope

Read these instructions carefully before using the HEINE mini 3000® instruments and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

#### Safety information

**Range of applications:** The HEINE mini 3000® dermatoscope is designed solely for the clinical examination of the surface of the skin.

**Safety in use:** Use only power sources which are designed for medical applications.

**Important: Do not use alkali manganese batteries with lithium as bulb life will be reduced.**

#### Instructions

- Moisten the affected skin with dermatoscopy oil, for instance with a cotton wool swab.
- Switch on the dermatoscope and place it gently over the lesion so that it is in the centre of the contact plate (1a).
- The examiner's eyes **should be as close as possible** to the eye-pieces (2). With the free hand adjust the focussing ring (3) until a clearly-focused image is obtained, (in most cases it is only necessary to set up the focus once).
- For general dermatological examinations, use the dermatoscope as an illuminated loupe **by removing the contact plate**.

#### Removing the contact plate

The contact plate (1a) is attached by a bayonet fitting. To remove, simply rotate the knurled ring counter-clockwise and detach from the dermatoscope. The small contact plate (1b) can be used instead of the contact plate (1a) for the examination of inaccessible lesions – see accessories. It is a simple push-fit in the head. To remove it, simply hold the knurled housing and pull off without twisting. When replacing, make sure that the light-guide (6) faces the bulb.

#### Changing the bulb

**Please note: The performance of this unit can only be guaranteed if genuine HEINE bulbs are used.**

- Detach the dermatoscope from the handle and remove the old bulb (4) by pulling.
- Clean the glass of the new bulb with a soft cloth.
- Push in the new bulb as far as it will go.
- The pin (5) must locate in the groove of the connector.

#### Cleaning and sterilising

The contact plate (1a) can be removed and sterilised or disinfected by any routine method e.g. in disinfectant solution, by washing, boiling or autoclaving (up to 134 degree Celsius for 5 min). The small contact plate (1b) and the dermatoscope can be cleaned by rubbing with alcohol or disinfectant. It should not be boiled, soaked or autoclaved.

#### Technical data

Magnification ca. 10x  
Focal range (correction) ca. ± 4 D  
Contact plate and small contact plate made from silica glass with internal anti-reflective coating.

#### Accessories and spare parts

Contact plate without scale	<b>D-000.78.116</b>
same, with scale	<b>D-000.78.117</b>
Small contact plate	<b>K-000.34.008</b>
Dermatoscopy oil (pack of 6 pcs.)	<b>K-000.34.005</b>
HEINE XHL halogen bulb 2,5 V	<b>X-001.88.109</b>

## Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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